


**Anmol Pulsar V - 2**  
**(Kumar's Magnet)**  
**Customer care: 9247110759**

This is a Magnet therapy device useful in Physiotherapy. To be used under the guidance of an experienced Magnet therapist or Physiotherapist or a Doctor.

This is not intended for primary medical care but only as a supportive one. Consult your doctor before using. Most of the models are specifically designed for individual usage, hence be careful while you get one for yourself.

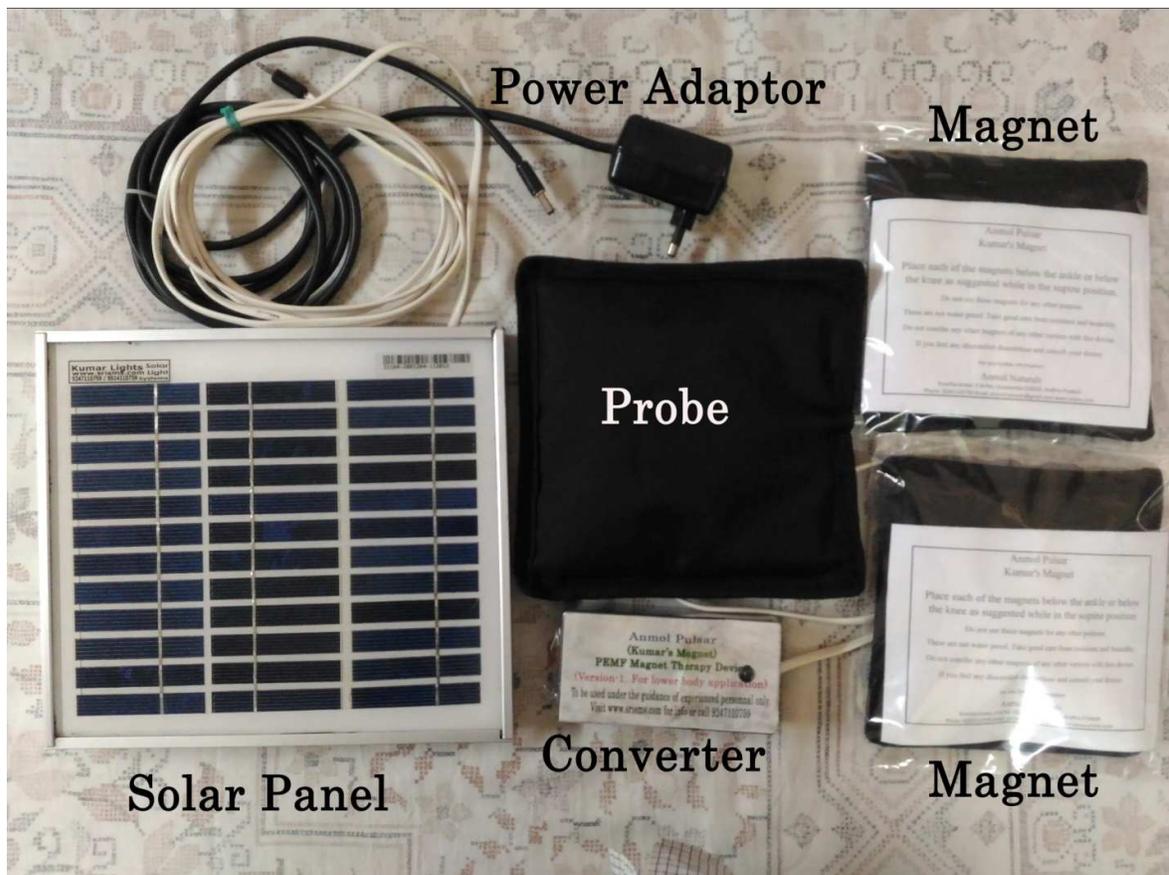

This Anmol Pulsar V-2 is designed to cater the needs of Diabetic care only. This helps in relieving issues related to diabetes and its allied disorders when used along with other / conventional therapy / treatment. The synergistic effect is observed to last more and comes in a rapid pace hence reducing the intake of medicines on tapering. This invisible and non-invasive magnet therapy is based on Bio-magnetism and gives better results.

Anmol Pulsar V-2 is the latest generation of Magnet therapy devices with PEMF (Pulsating Electro Magnetic Field) principles which has various power sources to give better relief at most times and places.

Anmol Pulsar V-2 is easy to use; no much prior training is required other than some mandatory cautions to be followed. Can be used for most of the people from ages 12 (twelve) and above.

Anmol Pulsar V-2 is dual powered with solar and grid (household) power. However it's much recommended to use on solar as the overall efficacy is found to be much higher than when compared to the same tenure of exposure taken at night or with grid power.

Anmol Pulsar V-2 works on Pulsating Electro Magnetic Field principles and brings the effect on the area it engulfs of the subject in treatment. The magnetic field only engulfs the area but the power is not induced forcibly into the body, hence it's much safer than other conventional TENS or similar type of devices. Though there are limitations for every device or concept this Anmol Pulsar V-2 surely is the less risk free option when handled by inexperienced or less knowledgeable persons.

**Precautions:**

1. Should be used under the guidance or in accordance with your family doctor or with a person who knows how to use this Anmol Pulsar V-2 safely and with cautions.
2. Should not be used during pregnancy
3. Should not be used for children below 12 years
4. Should not be used with persons having electronic implants
5. Should not be used with persons having braces and surgical implants
6. Should not be used during menstrual period, TB, viral diseases, internal hemorrhage, fevers, heart ailments, cancer etc.
7. Should not be placed directly near head as this is only for diabetic care.
8. Should not be used as a replacement for conventional medical treatment or as a reason to postpone seeing your doctor.
9. Should not be used when there is bleeding or on an undressed injury.
10. Should not be used during infectious diseases, TB etc
11. Consumption of magnetic water too should be limited to maximum of three times / day
12. Take all caution to place the head towards south or west sides ONLY while taking position.
13. If any sweating or irritation or any uncomfortable situation arises discontinue the usage and consult a doctor
14. A maximum of 20minutes exposure for a maximum of three times a day is necessary but in most cases just a 20 minute exposure once in a day is enough followed with same time of undisturbed movement of the body for the maximum effect to take place.
15. In most situations the blood circulation improves and people may get sleep during exposure / treatment, it is recommended not to disturb them at those times.
16. Taking of hot liquids is recommended before and after exposure / treatment and cold liquids / beverages SHOULD BE AVOIDED.
17. There should not be any electronic items like mobiles, tablets, computers etc within a vicinity of about 3 feet from the device. Since this Anmol Pulsar V-2 radiates low frequency emissions, it may harm other devices.
18. Keep the equipment out of reach of children
19. This unit contains a powerful magnet; hence see that any electronic or electromechanical devices are away from the device. They might get damaged permanently

20. If you are not sure where or how to apply contact customer care or contact your family doctor or contact anyone well versed with magnet therapy.

**Effective on:**

Issues related to diabetes, diabetic ulcers and other circulatory disorders which are in the lower part of the body.

**Specialties:**

Improves blood circulation, health and immunity, helps to normalize disorders of nervous and vascular system. Anmol Pulsar V2 helps to improve natural perspiration of the skin.

**Merits:**

Non-hazardous and no unpleasantness. No need for specially trained persons to handle.

**Model: Anmol Pulsar V2 Specifications:**

1. Solar panel of 12volts 4watts – 1no (model without solar panel is also available)
2. AC adaptor of 12volts – 1no
3. Energy convertor device– 1no
4. Probe – 1no
5. High power magnets in separate pouches – 2nos

**NON SOLAR MODEL DOES NOT INCLUDE SOLAR PANEL**

**Fixing and using the unit:**

If you are using the unit on solar mode, see that the solar panel is fixed in a well lit and sun exposed area and the solar module / panel is aptly fixed so that air / wind issues will not disrupt the panel position. See that the solar panel is fixed with a little inclination so that in the event of rain, fog etc the droplets will slide down easily. The best direction to fix the panel is south facing and 23 degree inclined. Then run the wire / cable attached to the panel to the place where the subject is in place. The solar panel cable has a pin, attach this to the converter unit. Once you connect the cable switch on the device. You should now see the indicator glows, if the probe is properly functioning you can feel a little vibration, which indicates that the unit is ready and functioning.

If you want to run the unit with household power with the adaptor, connect the adaptor to the wall socket and switch on the AC power. Connect this cable to the energy converter and switch on the device. You should now see the indicator glows, if the probe is properly functioning you can feel a little vibration, which indicates that the unit is ready and functioning.

Let the subject / person lie down on supine position & head towards south or west direction. Make sure he / she is not disturbed during the session. Maintain good ambience.

*South  
Or  
West  
Direction*



**Supine**

Once the power connections are set and the unit is working place the probe on the navel or below the navel (on the stomach) and the two magnet packs on /behind back or near kidney of the subject / patient. Keep the palms and feet separately, don't keep them crossed. After the session is over don't step directly on the floor, use foot wear.

**Duration of exposure / treatment:**

A maximum of 20 minute exposure is enough per sitting with a maximum of three sittings per day in equal intervals. In case of joint pains use of external medicated oils or any sort of rubefacient are an added advantage. See that the precautions are met without fail. Though the unit is water resistant (not water proof) see that no part comes in contact with water or any other sort of liquid. The Anmol Pulsar Version -2 does not come with any controls hence there is nothing to manipulate and is designed for care of diabetes.

**Applications of Anmol Pulsar V2:**

Issues related to diabetes, diabetic ulcers and other circulatory disorders which are in the lower part of the body.

**When to use:**

1. It is always better to take treatment / exposure before one hour of intake of food.
2. Try to maintain the same time everyday for better efficacy
3. Once a while treatment is not effective
4. Should not be applied near to head and heart, this is not designed to handle those issues.
5. After the sitting of 20minutes, its advised to take rest for another 20minutes without any movement
6. The best position to use is to place the probe below the navel point on the top / on the stomach in the supine position

**Observation during treatment:**

1. There might be some warmth and drawing sensation in limbs after 5-10minutes of the commencement of treatment. This happens only in initial stages that too only for some. If it's unpleasant withdraw the sitting and consult your doctor.
2. As the blood circulation starts improving, the treated may experience slight palpitation of heart. If this happens for a longer time discontinue the treatment and consult your doctor. The feeling of body becoming light and tendency to sleep may occur. For many the symptom of sleep continues.
3. After the sitting, the treated will feel rejuvenated and experiences activeness in routine life.

**Magnetized water:**

After getting the desired results, it is better to drink magnetized water at least once a day for a minimum of 15 to 20 days or every day ever after. Keep one or two ounces of fresh drinking water filled in a glass on the probe for 10 to 20 min and the same can be consumed. This magnetized water is very much helpful for proper maintenance of health. Do not store this water for a long time. It is to be consumed in 30minutes.

Magnetized water can also be used for household plants. This is particularly useful for creepers and crawlers and flowering plants.

**Troubleshooting:**

1. If the probe is not vibrating check the pin of the power input of the power converter. If the connections are okay then the power converter indicator too will light up or else the indicator will not light up.
2. If the indicator on the power converter lights up and only the probe is not vibrating then refer the unit to the supplier.
3. If using on solar panel mode see that the best time to operate is between 10am to 4pm
4. If you find any of the units (probe or the power converter) are hot which is an anomaly, refer to servicing
5. Do not use the adaptor given to any other device or for any other purpose.
6. Do not use the solar panel for other necessities.

Notice: All these devices are patent pending and all the instructions are given as per the findings from individual research and general data. Any infringement and usage of information provided and if the designs are copied will lead to legal complications. Use of this device is at your own advantage and discretion. To be used only upon advice of your family doctor or a registered medical practitioner or magnet-therapist.

Sold and serviced by:

## Sri Sresti Marketing

16-3-32/B, Kowtha Street,

Purnanandampet,

VIJAYAWADA – 520003. Andhra Pradesh.

Phone: 927110759

Email: [srestimarketing@gmail.com](mailto:srestimarketing@gmail.com) website: [www.srisms.com](http://www.srisms.com)